

PRIVACY FLAP (PFLAP) INSTALLATION INSTRUCTIONS



Our Privacy Flaps are designed specifically for use with all our standard sized Invacare-Matrix Backs. Each of our standard Invacare-Matrix Backs comes with a pre-sewn Velcro™ loop strip located across the bottom rear of cushion cover. One end of the Privacy Flap is secure to the Velcro™ strip on the back cushion cover, while the opposite end is secured to the wheelchair seat pan or sling seat. Privacy Flaps are available in 4 unique sizes (Xtra-Small, Small, Medium and Large). Flap sizes and dimensions are identified in the table below.

Privacy Flap (PFLAP) Sizing* (Width x Length)			
PFLAP-XS	PFLAP-S	PFLAP-M	PFLAP-L
(10"x16" 26cm x 41cm)	(13"x16" 33cm x 41cm)	(15"x16" 38cm x 41cm)	(17"x16" 43cm x 41cm)

*Note: A Privacy Flap Compatibility Chart is available (www.motionconcepts.com) to assist in determining the recommended PFLAP size for the full range of Invacare-Matrix Backs.



Installation Instructions:

1. Ensure the back is adjusted to the desired height, and remove the seat cushion from the wheelchair.
2. (Facing the rear of the wheelchair) Install the Privacy Flap (PFLAP) onto the Matrix Back by aligning the Velcro™ hook on the PFLAP with the Velcro™ loop sewn into the bottom rear of the back cover (see (1) and (2) below).
3. Pull the opposite end of the flap through the gap between the backrest and the wheelchair seat and secure the PFLAP (Velcro™ loop) onto the wheelchair seat* (3).

*Note: The type of Velcro™ pre-installed on the wheelchair seat pan/sling seat may vary. In order to secure the PFLAP onto the wheelchair seat, it may be necessary to install the Double-Sided Velcro™ Hook strip provided (4).

4. With the PFLAP in position, re-install the seat cushion on the wheelchair.



Invacare www.invacare.com



USA

One Invacare Way
Elyria, Ohio USA 44036-2125
440-329-6000
800-333-6900

Canada

570 Matheson Blvd. E Unit 8
Mississauga Ontario
L4Z 4G4 Canada
800-668-5324

Motion Concepts www.motionconcepts.com



USA

700 Ensminger Rd. Suite 112
Tonawanda, NY 14150
888-433-6818

Canada

84 Citation Dr., Unit 1
Concord, Ontario L4K 3C1
905-695-0134
866-748-7943



© 2012 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and ©. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.