


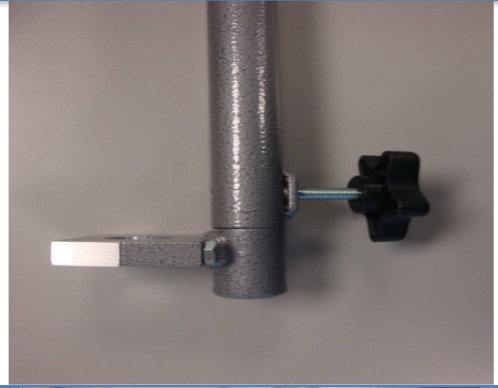


**Parts Required:** IV Pole & Holder

**Tools Used:** 7/16" Wrench

Step	Execution	Figures
<b>1</b>	Insert the two 1/4"-20 x 1-3/4" long hex cap screws, through the pass holes located on the mounting bracket.	
<b>2</b>	Place mounting bracket on one of the rear caster posts, just above the welded cross bars. Place the clamp on the opposite side of the caster post & thread the hex cap screws, installed in the step one, through the clamp. Use 7/16" wrench to tighten.	
<b>3</b>	Step 2 continued	

- 4 Once the mounting bracket is tightened, slide the larger opening ,of the IV Pole Holder, over the 7/8" stud on the mounting bracket. Use the 5/16"-18" Hand Wheel to secure.



- 5 At the top, swaged end, of the IV Pole Holder, insert the IV Pole. Insert the 5/16"-18 x 3/8" long set screw into one of the weld nuts, at the top of the IV Pole holder. Insert a 5/16"-18 Hand Wheel in the Adjacent Weld Nut.



6



**For more information, please contact Broda Customer Care at 1-800-668-0637**