






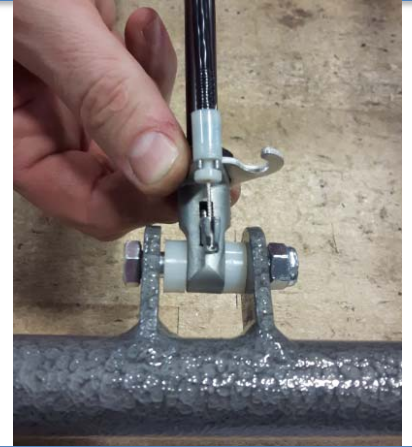
Parts Required: Cable x 2 (32", 48" or 56" cable lengths)
Handle

Tools Used: #2 Robertson Screwdriver
17mm Wrench

Step	Execution	Figures
1	Remove the cable tie holding the two cables to the tilt bar.	
2	Using a #2 Robertson screwdriver, remove the screw from the Tilt Handle.	
3	Using a 17mm wrench, loosen the black cylinder nut, on the bottom of both tilt cylinders. Note: The current style cable clip includes a washer, which does not require the nut to be loosened, please ignore this step and move onto step 4.	



- 4** Rotate the cable clip free from both cables. Remove the end of the cable from the actuator lever.



- 5** Using a #2 Robertson screw driver, attach the new Double Cable Handle to the recline bar.



- 6** Note: using cable ties, tie the cables on either side of the cylinders, securing to the back frame. See picture.




- 7** Attach the end of each cable to the actuator levers, the same way removed in step # 5.





- 9 Secure Cable Clips to the end of the Cables, attached to the actuator.

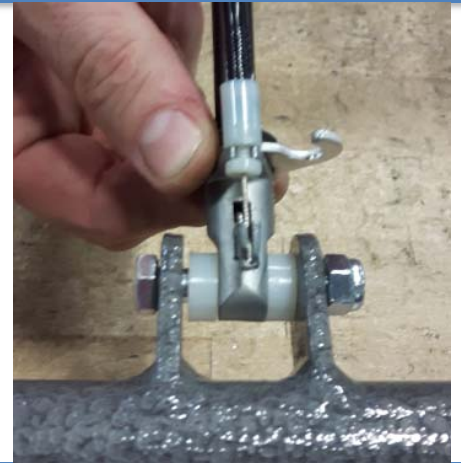

- 10 Using a 17mm wrench, tighten the black cylinder nut, on the bottom of each cylinder.

Note: The current style cable clip includes a washer, which does not require the nut to be loosened, please ignore this step.



For more information, please contact Broda Customer Care at 1-800-668-0637