

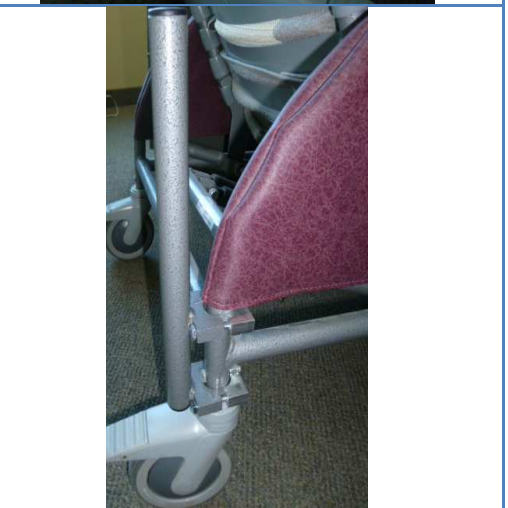


Parts Required: Adapter
Aluminum Clamps

Tools Used: 7/16" Wrench

Step	Execution	Figures
1	<p>The Adapter should be received as shown in the picture. You may need to unthread the screws attached to the block clamps and aluminum clamps, completely removing the aluminum clamps, before positioning on the chair.</p>	
2	<p>At the rear of the chair (left or right side), place the O2 holder attachment directly above the caster. Place the aluminum clamps on the inside of the caster post tube.</p>	
3	<p>Insert the bolts through the block clamps to the aluminum clamps and tighten, securing to the chair frame. Attach O2 Holder to adapter (see instructions). Note: depending on the location the adapter is mounted on the Pedal or Midline chair, this may slightly limit some of the tilt and/or recline functions.</p>	

For more information, please contact Broda Customer Care at 1-800-668-0637