


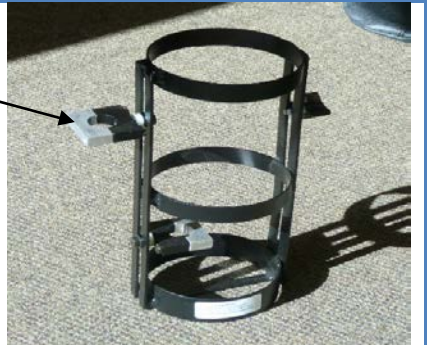

- Parts Required:**
- O2 Holder
 - IV Bracket
 - 1" Block Clamp
 - O2 Mount Adapter

- Tools Used:**
- 7/16" Wrench

Step	Execution	Figures
1	<p>Oxygen Tank Holder will arrive with clamps attached on the inside. The bolt will need to be removed, and clamps switched to the outside.</p>	
2	<p>Mounting the O2 holder to the 587 V3 will require an IV Bracket and clamp, which you will receive in the O2 Holder kit.</p>	
3	<p>Mount IV Bracket on the 587V3 base frame, right behind the front caster post. Secure the IV Bracket, by mounting the block clamp onto the opposite side of the tubing.</p>	

- 4 Once the IV Bracket is secure, slide the O2 Mount Adapter, over the IV Bracket so that the holes line up. Secure with a 1/4-20 x 1-1/2 Hex Cap Screw & Nylon Insert 1/4-20" locknut.



- 5 Mount the O2 Holder to the O2 Mount Adapter, using the black clamps already attached to the O2 holder.



- 6 Secure the O2 Holder to the adapter, by mounting the block clamps on the opposite side of the adapter tube. Secure with the 1/4"-20 x 1-3/4" hex cap screw.





For more information, please contact Broda Customer Care at 1-800-668-0637